## DEUTSCH

## ENGLISH

### A VOREINSTELLUNGEN

### A PRE-SETTING

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#### 1. Wheelchair mounting

Fixation of cramps in an optimal position for the user wheelchair.

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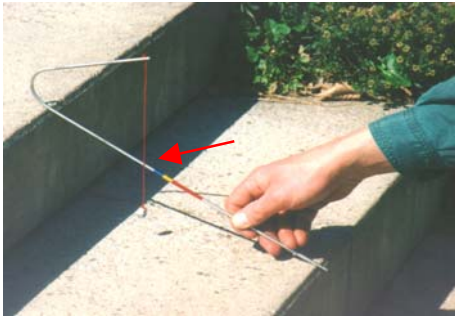
#### 2. Adjustable safety rails

The adjustable safety rails are serving to prevent dumping downstairs. The measuring pendulum indicates the necessary assembly position:

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**Blue:** approx. 33 - 36,5°  
**Yellow:** " 36,5 - 40°  
**Red:** intolerable area!  
**Colourless:** safety rails not required

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Above pre-adjustments are only to be initiated by trained personnel. They have to be tested practically in any case. For this purpose precaution measures have to be taken!

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The pre-setting and a testdrive have to be repeated whenever one of the decisive parameters are changing: the staircase, the wheelchair, the passenger.

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#### 1. Positioning of the adaptor

The adaptor has to put into the rail-track and to be kept upright.

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The wheelchair is positioned in the middle of the machine and can be rolled backwards with the brakes released.

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#### 2. Fastening of the wheelchair

Catch the backpipes with the cramps (using the pivoted lever for width adjustment), pull and move up the ball-grip until the cramps are touching the wheelchair handles. Then let the ball-grip engage into the next hole below. Check now for an impeccable fitting of the cramps bothsides.

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The safety belt is to put below the arms of the passenger. Adjust the headrest.