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3. Positioning into drive position

Bend the adaptor including the wheelchair backwards until all 4 white rolls are placed into the glide rail. After you have to pull the adaptor backwards until the locking bolt clicks in. Several safety equipments are preventing to drive in non-locked position, just as an opening the locking bolt isn't possible during the stair-ride.

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4. Put into operation

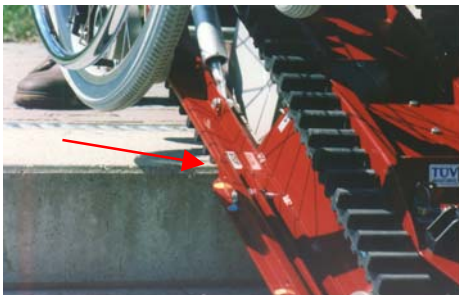
Key switch to "ON" and turn the handles in the desired driving direction (engaging position). Only by continuing to turn the handle, the machine will be put into operation. The built in springcontrol is interrupting the journey after loosening of the handle.

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Attention: Do not overturn by force the switch!

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Important: All stairs as well as single steps have always to be overcome with your back being positioned in direction to the stair, whether you go upstairs or downstairs!

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5. Moving at the top of the stair

If the "STOP" mark has reached the top level of the stairflight (due for up- or downwards trip), the journey is to be stopped and the machine has to be turned from bevel of stair into the podest (or inverted), after that only the trip can be continued:

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6. Moving on flat ground

By slightly pushing down (not lifting!) of the handles the tracks will lift from ground, herewith any turning and driving manoeuvres easy can be effected without motorpower.

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7. Releasing of the wheelchair

Turning grips in position "0". First pull back the adaptor while you put the foot on the trackband. This sets free the safety nose of locking bolt. Now lift the handlever and let roll the adaptor to the front, lift the adaptor so that the wheelchair comes on its wheels. Open the safety belt, pull the ballgrip, let glide the mounting downwards and open the cramps by the pivoting lever.

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